


Stainless Steel Wheelchair

GENERAL WARNINGS

- 1) BEFORE USING THIS WHEELCHAIR, READ THIS MANUAL AND SAVE FOR FUTURE REFERENCE.
- 2) ALWAYS INSPECT THE WHEELCHAIR TO ENSURE THAT IT IS PROPERLY POSITIONED AND STABLE BEFORE USING. **DO NOT** USE IF WOBBLY OR UNSTABLE.

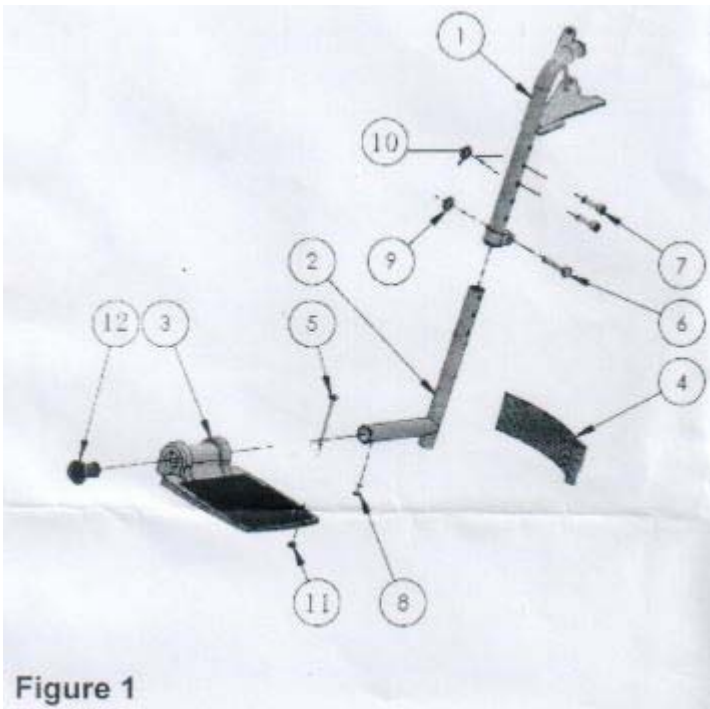

ITEM #	QTY	DESCRIPTION
1	1	UPPER FOOTREST WELDMENT
2	1	LOWER FOOTREST WELDMENT
3	1	FOOTREST PLATE
4	1	FOOTREST STRAP
5	1	FOOTREST PIN
6	1	M8X30 SCREW SS
7	2	M6X30 SCREW SS
8	2	5X12 RIVET
9	1	M8 NUT SS
10	2	M6 NUT SS
11	1	M6 NUT SS
12	1	PLASTIC CAP, BLACK

ADJUSTING FOOTREST HEIGHT (REFER TO FIGURE 1)

- 1) USING THE TOOL PROVIDED, UNSCREW THE NUTS (ITEMS 9 & 10) AND REMOVE THE SCREWS (ITEMS 6 & 7).
- 2) PULL LOWER FOOTREST WELDMENT (ITEM 2) DOWNWARD; ADJUST TO SUITABLE POSITION.
- 3) REPLACE THE SCREWS AND NUTS AND WRENCH TIGHT TO SECURE (ITEMS 6, 7, 9 & 10).

NOTE: MAKE SURE THE TWO PEDALS ARE ADJUSTED TO THE SAME HEIGHT AND ENSURE THE BOTTOM FOOT SCREW IS SCREWED TIGHT.