



mobi
personal transporter

DELUXE MODEL

INSTRUCTION GUIDE



**COMPLETE BALANCE
SUPPORT**



**COMFORTABLE
TEMPORARY SEAT**



HEIGHT ADJUSTABLE



**300 POUND
WEIGHT CAPACITY**

**ALL INSTRUCTIONS ENCLOSED MUST
BE READ, UNDERSTOOD, BEFORE USE.**

walk safely.
walk in style.

DELUXE PERSONAL TRANSPORTER



mobi
personal transporter



CONTENTS:

Carefully remove the Personal Transporter frame and parts from the box and inspect them for any damage or missing parts that may have resulted from shipping.

PARTS INSIDE BOX:

- (1) Personal Transporter Frame with Bag and Seat (See image 1).
- (1) Padded Backrest (see image 2).
- (2) Front Leg-Wheel Part, marked 'L' for Left and 'R' for Right (see image 3).
- (2) Rear Leg-Wheel Part with Cable, Handle and Hand Brake, marked 'L' for Left and 'R' for Right (see image 4).
- (4) Leg-Wheel Adjustment Knobs - longer, and (2) Handle Adjustment Knobs - shorter (see image 5).





ASSEMBLY INSTRUCTIONS:

1. OPEN THE FRAME

- Place frame on the floor with top of seat facing up, (see image 6).
- Spread frame out, lift up padded seat and press down on bottom of storage bag. User will feel folding bar. Push down on folding bar until it locks in place, (see image 6).

2. INSERT PADDED BACKREST


- Insert Padded Backrest into the plastic backrest hinges, (see image 7).
- Using thumb, press down on the push button and slide the Padded Backrest into the hinge until the buttons engage into the hinge holes. Ensure that the push buttons fully engage in the hinge holes.
- Pull up on the Padded Backrest to ensure it is properly installed
- **Note:** Padded Backrest can be folded up when not in use

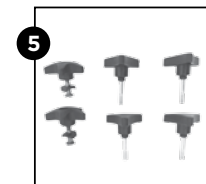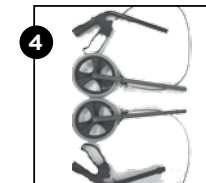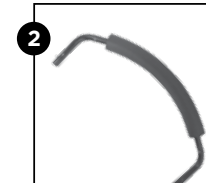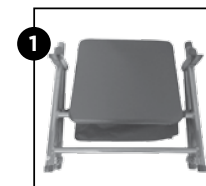



3. INSERT FRONT WHEELS

- Turn Personal Transporter frame over so that it rests on it's back (Mobi logo on front of padded seat should be facing up), (see image 8).
- Insert the right front Leg-Wheel Part labeled 'R' into the frame tubing so that it matches up with the letter 'R' on the frame leg, (see image 9).
- Press down on the push button and slide the Leg-Wheel Part to your desired height position, (see recommended seat height below).
- Repeat process, above, for left front Leg-Wheel Part, labeled "L".
- Make sure that the push button is fully engaged in the hole for both Leg-Wheel Parts and that the legs are even.
- Once desired height is achieved, insert one of the (4) Leg-Wheel Adjustment Knobs, threading from the outside for both leg-wheel parts, (see image 10). Tighten firmly.

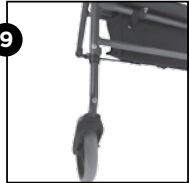

4. ATTACH REAR LEG-WHEEL PARTS (WITH BRAKES):

- With Personal Transporter still resting on it's back, insert the right rear Leg-Wheel Part, (with handle and brake), labeled 'R' into the frame tubing so that the Leg-Wheel Part matches up with the letter 'R' on the frame leg, (see image 11).
- Press down on the push button and slide the rear Leg-Wheel Part to your desired height position. Refer to Recommended Seat Height Instructions, next section.
- Insert one of the (4) Leg-Wheel Adjustment Knobs, threading from the outside, (see image 10). Tighten firmly.
- Repeat above steps to attach the second rear Leg-Wheel Part (with handle and brake), (labeled L).
- Turn Personal Transporter right side up. Make sure the Personal Transporter is level and that all four Leg-Wheel Parts are in the same position on the frame, and that the (4) Leg-Wheel Adjustment Knobs are **firmly tightened**.



5. RECOMMENDED SEAT HEIGHT:

- The user's feet should be flat on the ground when properly seated on the Personal Transporter.
- There are three height adjustment positions on the Mobi Personal Transporter
- Seat height on the Personal Transporter is determined by the positioning of the Leg-Wheel Parts, front and back, on the frame.
- The first position, as the Leg-Wheel Part engages the frame leg, is the tallest position, for taller people.
- The second position, as the Leg-Wheel Part enters the frame leg, is the middle position, for people of medium height.
- The third position, as the Leg-Wheel Part enters the frame, is the last position, for people of shorter height.
- Make sure to test different positions to find the proper height adjustment.
- Make sure that personal transporter is level after adjustments



6. INSERT HANDLES:


- Take Handle attached to "R" Leg-Wheel Part and insert handle tube into frame by sliding into open tube in frame, (see image 12).
- To secure, thread one of the (2) Handle Adjustment Knob **bolts** from the **inside** of the frame, **outward**. The bolt head rests in an opening of the same shape. Bolt head should be flush with metal tube. Insert plastic washer and Tightening Knob onto bolt from outward side of tubing. Tighten firmly, (see image 13).
- Repeat above steps to insert second, "L" Handle.

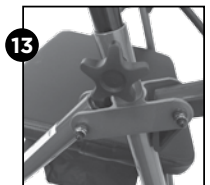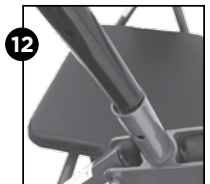



7. DETERMINING HANDLE HEIGHT:

- User should stand behind Personal Transporter with their arms hanging at their side and both feet lined up with the rear wheels, (see image 14).
- Personal Transporter handles should be even with the wrist of the user.
- Adjust Handle Tube positions accordingly, per Insert Handles instructions above.

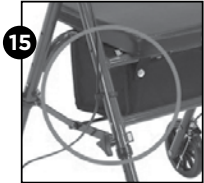

8. CABLE CLIP

- Personal Transporter comes with two Cable Clips to keep brake cables secured to side of unit, (see image 15).
- Make sure that both brake cables are connected to the Cable Clips on both front left and front right frame sections, (see image 15).



OPERATING INSTRUCTIONS:

1. TO OPEN PERSONAL TRANSPORTER


- Place unit upright on floor.
- Lift up Padded Seat and press down on bottom of Storage Bag. User will feel Folding Bar. Push down on Folding Bar until it locks in place, (see image 16).

2. TO CLOSE, FOLD PERSONAL TRANSPORTER

- Lift up on Padded Seat.
- Lift up on the nylon handle at bottom of Storage Bag. This engages the Folding Bar, (see image 16).

3. BRAKES



- Squeeze the brakes to stop the Personal Transporter, (see image 17).
- Push the brakes straight down to the locking position, (see image 18).

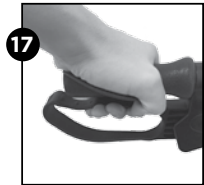

- BRAKES MUST BE IN THE LOCKED POSITION WHENEVER SITTING ON THE PERSONAL TRANSPORTER. ALWAYS ENGAGE, DISENGAGE BRAKES SIMULTANEOUSLY.

4. CHECK HAND BRAKES

- Hand Brakes should be in working order after assembly
- Check Hand Brakes periodically for proper function
- Squeeze the Hand Brakes and check to ensure that the metal brake shoes engage the wheels and stop the Personal Transporter, (see images 19 and 20).
- Push the Hand Brakes DOWN into their locking position and check to ensure that the metal brake shoes engage the wheels and stop the Personal Transporter, (see images 19 and 20).

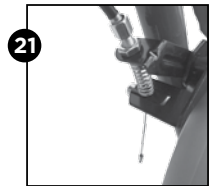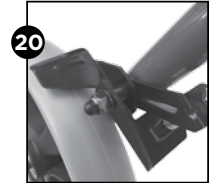

5. HAND BRAKE ADJUSTMENT

- Hand Brakes should be in working order after assembly.
- However brakes can loosen during normal use

TIGHTEN BRAKES

- Place Personal Transporter upright on floor
- Loosen nut at top of brake shoe assembly, near wheel. Turn **counter-clockwise** until desired tension is achieved, (see image 21).
- Test by squeezing hand brake — ensure that metal brake shoe fully engages the wheel.

LOOSEN BRAKES

- Place Personal Transporter upright on floor
- Loosen nut at top of brake shoe assembly, near wheel. Turn **clockwise** until desired tension is achieved, (see image 21).
- Test by squeezing hand brake — ensure that metal brake screw fully engages the wheel.



SAFETY PRECAUTIONS:

1. READ ALL ASSEMBLY, OPERATING INSTRUCTIONS BEFORE USE
2. Please make sure Personal Transporter is fully opened, seat is down and the Folding Bar under seat is pushed down, locked into position
3. Make sure all knobs, nuts, bolts are fully tightened. Check regularly to ensure they are secure at all times
4. Make sure Leg-Wheel Parts are in same height position; Personal Transporter is level
5. Test Personal Transporter for stability before applying full weight to hand grips
6. Test brakes before using
7. ALWAYS LOCK BRAKES BEFORE SITTING, please refer to Operating Instructions, #3 Brakes and Image 18
8. Replace tires when significantly worn or damaged
9. This Personal Transporter is a walking aid only. **It is not designed as a Transport Chair or Wheelchair and therefore is not intended for one person to push another while seated**
10. Caution: Keep hands clear of frame when folding

JUVO PRODUCTS, LLC PERSONAL TRANSPORTER, ROLLATOR LIMITED LIFETIME WARRANTY

Juvo takes great pride in providing products of superior quality, design and function.

This Limited Lifetime Warranty applies to Rollator, Personal Transporter products manufactured by or on behalf of Juvo Products, LLC, (Juvo), under the brand name "Juvo®" or "Mobi™," and referred to in this Limited Lifetime Warranty as "Product."

Product is warranted to be free from defects in materials and workmanship for so long as the original owner owns such Product. In the event of any such defect during the warranty period, Juvo® will remedy the defect as described herein, without charge to the purchaser, as soon as practicable. The remedy will consist of repair or replacement of the Product, or refund of the purchase price, at the discretion of Juvo® after opportunity to inspect the alleged defect or photographs thereof. All freight and labor charges will be the responsibility of the purchaser.

This Limited Lifetime Warranty applies to the original owner only and is non-transferable. This Limited Warranty does not cover:

- Labor for any inspection.
- Labor for replacement of any defective item(s).
- Incidental or consequential damages.
- Minor cosmetic defects, which do not affect performance or integrity of a part or the entire Product.
- Non-durable parts that are subject to normal wear and tear, such as Wheels, Hand Grips, Padding, Cables-Brakes, Upholstery, Bag, Plastic Parts.
- Vandalism, improper use, or negligence of the owner.
- Scratches or scrapes to materials or parts, dents, tears in fabric, or other issues that arise out of normal use of the Product.

This Limited Lifetime Warranty applies to Product purchased for personal use only. Product purchased for institutional multi-use environments are excluded from this warranty.

Juvo® Products have been designed for safety and quality. Any modifications made to the original Product could damage the structural integrity of the unit leading to failure and possible injury. Modification of the original Product voids any and all warranties and Juvo® assumes no liability for any modified Products or consequences resulting from failure of a modified Product.



Warranty Claims: The purchaser must notify Juvo directly of any claims under this Limited Lifetime Warranty within seven (7) days following the discovery of the problem. To make a claim under this Limited Lifetime Warranty, the Buyer should contact Juvo Products, LLC at:

**111 North Street, Suite 205
Barrington, Illinois 60010
Attention: Customer Service**

TOLL FREE CUSTOMER SERVICE: 844.277.6191

A copy of the original purchase receipt should accompany any claim under this Limited Lifetime Warranty.

Juvo shall have no obligations under this Limited Lifetime Warranty if the purchaser fails or refuses to reasonably cooperate in Juvo's investigation of the claim (such as failing to provide photographs and or written documentation of the claim). If Juvo determines there are defects covered by this Limited Lifetime Warranty, Juvo will have up to thirty (30) days after receipt of notification to process the purchaser's claim.

This Warranty gives you specific legal rights. You may have other rights as well which vary from state to state or province to province. This warranty excludes all consequential damages, however, some states do not allow the limitation or exclusion of consequential damages and therefore this limitation may not apply to you.



LIFESTYLE PRODUCTS

that improve mobility and safety
around the home



BALANCE & MOBILITY



OPENING



DRESSING



REACHING



SAFETY



TOILETING



BATHING



CARRYING



SURGERY RECOVERY



**SEE ALL JUVO LIFESTYLE PRODUCTS
AT JUVOPRODUCTS.COM**

Made in China



mobi

personal transporter

by

live actively
Juvo[®]

111 North Street, Suite 205
Barrington, Illinois 60010

844.277.6191 TOLL FREE

juvoproducts.com