

## COMFY-SEAT

These padded liners attach easily with straps to wheelchairs or geri-chairs. They help prevent and eliminate pinch points and drafts, while keeping skin from sticking to vinyl seats and backs. Both models zip apart for easier laundering, and the wheelchair model has side zippers to allow removal of the wheelchair arms, if desired.



## MONITORING

**⚠️WARNING** Be sure to follow your facility's policies and guidelines for frequency of patient monitoring.

## INSPECTION

**⚠️WARNING** Inspect before each use: check for broken stitches or zippers; or torn, cut or frayed material. DO NOT use soiled or damaged products.

## APPLICATION INSTRUCTIONS:

1. Fit Comfy Seat over the backrest and seat.
2. Attach to the wheelchair/geri-chair by affixing straps in the back and securing ties over the armrest and wheelchair frame.

MODEL	SEAT	BACK	SIDE PANELS
<b>REF 6526</b>	20"W x 18"D 51 cm x 46 cm	20"W x 17"H 51 cm x 43 cm	15"D x 14"H 38 cm x 36 cm
<b>REF 6526G</b>	22"W x 20"D 56 cm x 51 cm	22"W x 31"H 56 cm x 79 cm	20"D x 20"H 51 cm x 51 cm

## LAUNDERING INSTRUCTIONS:



## DISPOSAL

**⚠️WARNING** Properly dispose of the product per facility's policy for BIOHAZARDOUS materials.

## STORAGE AND HANDLING

This device is designed for use in normal indoor environments. This device may be stored in ambient warehouse temperatures at normal humidity levels. Avoid excess moisture or high humidity that may damage product materials.

