



Convenient Carry Case

Re-Order Number	Description	Size	Units
703080	Wheelchair Rain Cape	Universal	Each

Purpose

The Skil-Care All Weather Wheelchair Cape fits easily over a person sitting in a wheelchair without having to remove them. The waterproof and wind resistant fabric will keep the person dry and warm.

Use the convenient carry case supplied to store the All Weather Wheelchair Cape on the wheelchair in case of sudden inclement weather.

Easy to Use

Before placing the rain cape on, make sure that the smooth vinyl fabric is on the outside. Place the rain cape on with the hands through the arm cut outs and the head placed in the hood. When sitting in the wheelchair place the cape between the arm rests and body. Do not place the rain cape over the wheels as the cape may get caught in the wheels and will soil.

Easy Care

This material is wind and water resistant and may be cleaned with mild detergent. It may also be hand washed and air dried. Do not launder.

This product is guaranteed for 6 months.