

Made in U.S.A.

GREAT STUFF!
Scooters
Wheelchairs
For & Walkers



Diestco Manufacturing Corporation • P.O. Box 6504 • Chico, CA 95927 • 1-800-795-2392

Folding Manual Wheelchair Cover

Part No. V1200

Now with 300 yard visible reflective safety tape!!



400-denier nylon with a urethane backing provides the highest level of protection from the elements for your wheelchair. Works great on lifts or for just storing outside.

Products Proudly Made in the U.S.A.


 **DIESTCO Mfg. Corp. • 800-795-2392 • www.diestco.com**

Folding Manual Wheelchair Cover

The cover will provide protection for your vehicle from the weather and dust. Yet, it is lightweight and easy to get on and off!

Slip the cover over your vehicle and pull the elastic down under your chassis. Pull the cord lock so that it is tight around the bottom. For extra security, when transporting your vehicle on the back of your car or van, tie down the cover by hooking bungee-type cords (not included) over the top of the cover and secure at the bottom.

Note 1: This is a Standard Cover; you will have grommets with a draw cord and lock to tighten.

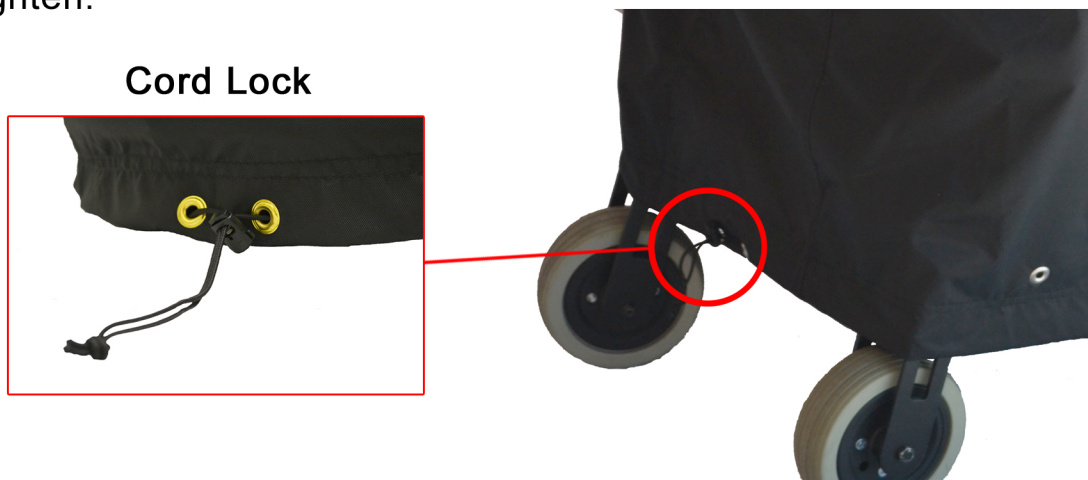

Note 2: Depending on your vehicle, cover may not fit over wheels.

Note 3: The cover is not intended for highway use without using some securing device (i.e. rope or bungee cords) over the top of the cover. This will reduce the amount of excess room under the cover and prevent the cover from billowing up at high speeds. **If your cover is flapping excessively at high speeds, the life of the cover will be greatly reduced.**