





**INSTRUCTIONS FOR INSTALLING YOUR NEW  
VINYL ELECTRIC WHEELCHAIR CONTROLLER UNIT COVER**


**1**  To install the Wheelchair Buddy™ Power Control System Cover, remove it from the packaging and view the picture carefully. Loosen the Velcro.

**2**  Slide the cover over the top of the unit so that it appears as in the picture when viewed from the bottom.

**3**  Now slide the cover down towards the joystick and raise the rubber seal. Slide the cover under the seal and around the joystick and give a gentle tug. Close the long Velcro strips and reset the rubber seal back in place.

**4**  Now carefully look at the picture. Take one hand and while you hold the locking mechanism, press firmly on locking device and pull the elastic cord so it appears as in the picture. (See Arrow).

**5**  Close the two wings making sure that the Velcro is firmly together.

**6**  Inspect the installation. It should appear as in the pictures. You are done!

*And now we proudly welcome YOU into the Wheelchair Buddy™ Family.*

**Congratulations!**

**Wheelchair Buddy**

**VINYL ELECTRIC WHEELCHAIR  
CONTROLLER UNIT COVER  
BY THE OMNI ONE GROUP**

716 HANSEN AVENUE, LYNDORA, PENNSYLVANIA 16045

[www.thewheelchairbuddy.com](http://www.thewheelchairbuddy.com)

