

**Gel
Ovations™**



Rollator (4-Wheeled Walker) Handle Cover

Gel for Comfort

Zippers over existing handles to add comfort for 4-wheeled walkers having orthotic-style (wider) handles.

www.gelovations.com



- Improves comfort and decreases pressure
- Highest quality products on the market
- Easy to install – no tools required
- Washable neoprene/nylon non-slip cover
- Latex free

Installation Instructions

This product is designed to fit right over the existing handle grip of 4-wheel rolling walkers (Rollators) with the 'shaped orthotic grip'. For rolling walkers with straight grip handles use the Walker Handle Cover (WHC5).

Open the Gel Ovations™ Rollator Handle Cover unzipping the product. Stretch the product with the gel on top, shiny side against the existing handle, and so that the closed end is up against the end of the handle. Pull the zipper together and zip it closed so that it stretches tight around the handle.

Cleaning

Do NOT put pads/covers in a washing machine.

Do NOT submerge them in water or agitate.

Do NOT put in clothes dryer.

With HEATED DRYING turned OFF, you may wash them in the top rack of an automatic dishwasher.

Alternatively, wipe the surface with any normal household soap or cleaning solution that is not irritating and to which you are not allergic.

Limited Warranty

Gel Ovations™ warrants that this product will be free of manufacturing defects for a period of 12 months. If defective, please return to place of purchase for an exchange or full refund. This warranty does not extend to abuse of the product such as tampering or modification.

**Gel
Ovations™**

www.gelovations.com



Product Code: **RHC**