

The AbleTripod® is designed to fit all cane and forearm crutches.

Including: adjustable metal and folding, wood, & plastic.

If it appears you will damage the base while assembling, then you have purchased the wrong size.

Do Not Force Cane into Hole.

Contact Customer service if there are any questions about the fit (888) 988-2263.

How to Assemble to Your Cane:


1. Remove existing rubber tip
2. Place end of cane shaft into hole



3. Step on base - Firmly push down while twisting cane back and forth until properly seated at bottom of hole.
4. Cane must stand up straight with handle aligned over longer back POD (See Orientation illustration Below).

Correct AbleTripod® Orientation


Check orientation regularly especially when used with folding canes that tend to change alignment each time assembled.

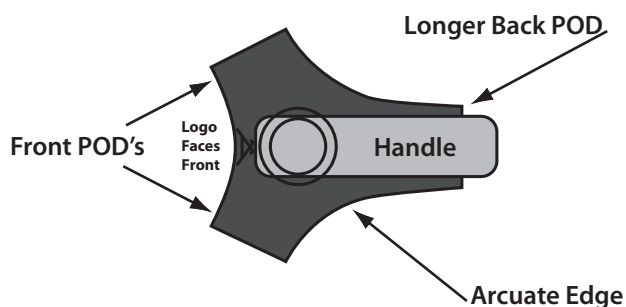

Adjust Your Cane Height:

(Adjustable Aluminum Cane Shown)
(See Information below for your cane type)

The proper height of your cane handle should align with your wrist while standing erect with your hand at your side

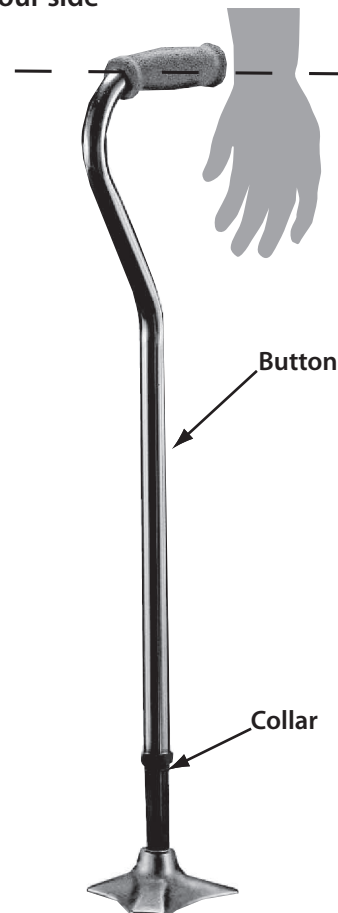

Cane Types recommended for AbleTripod® Base tip replacement:

Adjustable Canes (Shown)

- Loosen the collar.
- Depress button sliding cane up or down to adjust.
- Make sure button locks in place.
- Tighten collar

Straight Canes (Not Shown)

- Place handle end of cane on floor.
- Mark shaft at correct height to your wrist.
- Cut shaft square. (prevents cane from leaning)
- Follow assembly instructions

Forearm Crutches (Not Shown)

- Loosen collar or pull collar locking pin.
- Depress button and / or slide shaft to correct height position recommended by your care professional.
- Replace pin or make sure button locks in place, tighten collar (if applicable)
- Rotate AbleTripod® base so front pod, arcuate edge, and back pod share contact with floor