


Assembly Instructions For CS1200

As you unfold
TO₂TE you will
notice three (3)
Velcro® straps:

- A** Safety Strap Buckle
- B** Stabilizers (2)
- C** Centering Strap

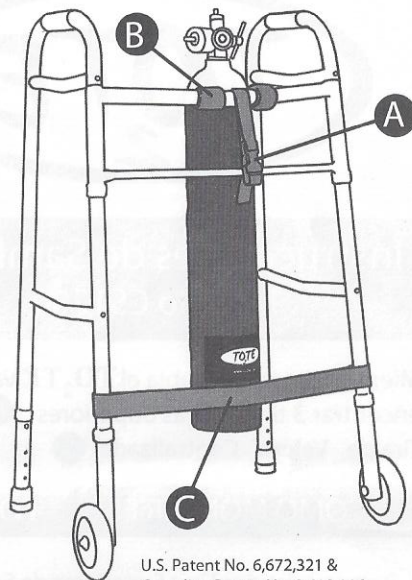

U.S. Patent No. 6,672,321 &
Canadian Patent No. 2,410,116

HOW TO ATTACH THE TO₂TE

- Put the **TO₂TE** on the **inside** of the walker and attach the Safety Strap Buckle **A** around the top bar of the wheeled walker, as shown. This safety strap should also go around the second bar, if your walker is so equipped. Next, place oxygen cylinder in **TO₂TE** for ease of centering.
- Loop the Velcro® Stabilizers **B** around the top bar of the walker and tighten evenly.
- Attach each end of the Centering Strap **C** around the left and right legs of the walker, as shown in the diagram. Fasten the Velcro® ends in the back and tighten evenly to center the oxygen cylinder.

When finished the **TO₂TE** should be centered, on the **INSIDE** of the walker, as shown in the diagram.

HOW TO REMOVE THE TO₂TE

Reverse the procedures above, removing the Safety Strap **A** last.

Washing instructions are inside of **TO₂TE**

WARNINGS!

This product was designed for use on a two wheeled walker only. Consult with your health care provider before using the **TO₂TE** on any wheeled walker.

Caution must be used with a wheeled walker when using the **TO₂TE**. This product is not intended for individuals who have difficulty with balance.

Follow installation instructions, make sure the **TO₂TE** is installed on the inside of the walker in the center, as shown in the drawing above. Failure to do so could shift the center of gravity resulting in injury.


Make sure the **TO₂TE** Velcro® Top and Centering Straps are not worn, ripped or broken. Check them frequently. Replace the **TO₂TE** if the straps are worn, ripped, or broken or if the straps appear to be damaged in any way. Failure to do so could result in injury.

If you find that you need assistance installing this walker accessory, Email: support@to2te.com.



Assembly Instructions For CS1200

As you unfold the **TO₂TE** you will notice 3 straps:
2 Top Straps **A B** Centering Velcro® Strap **C**

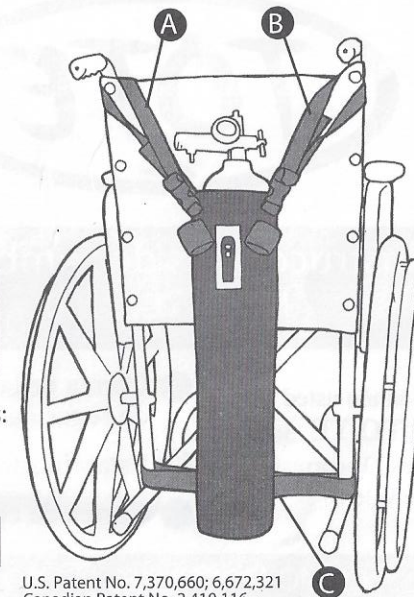

U.S. Patent No. 7,370,660; 6,672,321
Canadian Patent No. 2,410,116
& Foreign Patent Pending

HOW TO ATTACH THE TO₂TE

- Remove two Top Support Straps left and right from Velcro® loops.
- Place the **TO₂TE** on the back of the wheelchair and attach the 2 Top Straps **A & B**. Place one strap **A** over the left handle and one strap **B** over the right handle and adjust up or down (position and height) as shown in the diagram. *Note the Wrench Storage Pocket faces out for easy access to valve wrench.*
- Place oxygen cylinder into **TO₂TE** and attach each end of the Centering Strap **C** around the left and right chair support bars starting from the outside and wrapping in, as shown in the diagram. Fasten the Velcro® ends and secure evenly to center the cylinder. To shorten the Velcro® straps fold each end over itself to the correct length for your wheelchair and then securely fasten. This Centering Strap has been specifically designed for this purpose.

HOW TO REMOVE THE TO₂TE

To remove the **TO₂TE** reverse the steps above. Washing Instructions are on the inside of **TO₂TE**.

WARNINGS!

Follow installation instructions; make sure the **TO₂TE** is installed exactly as shown in the diagram above.

Make sure the **TO₂TE** Velcro® Top and Centering Straps are not worn, ripped or broken. Check them frequently. Replace the **TO₂TE** if the straps are worn, ripped, or broken or if the straps appear to be damaged in any way. Failure to do so could result in injury.

If you find that you need assistance installing this wheelchair accessory, Email: support@to2te.com.



Health Care
Products
that make life a
little easier