




Folding Safety Flag

Mounting the flag is simple. This flag can mount just about anywhere on the vehicle. Kit includes: 2-1",2-3/4" square clamps, 2-7/8" round clamps and 2-1/4X1 bolts with lock nut. (note not all hardware is needed for mounting the flag)

Below is a few of the many ways that the flag can be mounted to the vehicle:

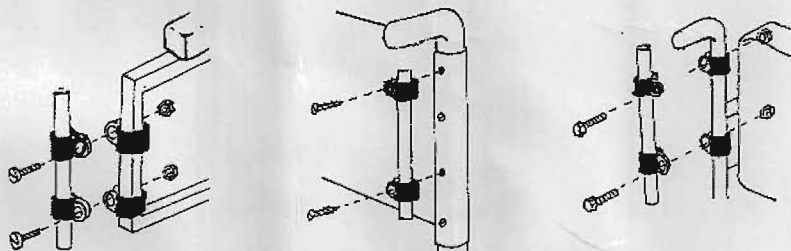

After the mounting hardware is in place insert the flag pole and push down to secure. The folding flag can be removed without taking off the mount.

We strongly encourage you to call us at (800) 795-2392 if you have any problems, questions, or comments. We are always in development of new products for wheelchair, scooter, crutch, and walker users...so let us know if there's something special you need!